

## IMPLANT COMPATIBILITY CAUTION HIP SYSTEMS:

STEMS, FEMORAL HEADS AND DISTAL CENTRALISERS ARE UNIQUE TO EACH SYSTEM.

- Stems and distal centralisers cannot be mixed and matched between our hip systems or any other manufacturer's systems. For example, Edinburgh™ must ONLY be used with Edinburgh™, Classic Marathon™ with Classic Marathon™ etc.

### Stems and femoral heads are unique to the extent of:

Where stem trunnions are either 10/12 or 12/14, they can be used with a modular head with a 10/12 or 12/14 cavity.

#### **Raw Material**

- Our stainless steel stems can **only** be used with our stainless steel heads or ceramic heads.
  - Our cobalt chrome stems can **only** be used with our cobalt chrome heads, titanium heads or ceramic heads.
  - Our titanium stems can **only** be used with our cobalt chrome heads, ceramic heads or titanium heads.
- Under **no circumstances** are our stems to be used with another manufacturer's femoral heads.
- If the indication demands the use of a hybrid 2-piece acetabular cup system, then our stem/head sets may be used with any of our 2-piece systems. Under special circumstances, our stem and distal centralisers and head sets may be matched with another manufacturer's 2-piece acetabular system where our 2-piece systems are not considered appropriate e.g. reconstructive revision acetabular systems. However, in these circumstances, please ensure that our 22.225mm Ø, 26mm Ø, 28mm Ø and 32mm Ø, or whichever diameter chosen, is matched with equivalent a cup and matching cavity.

#### **KNEE SYSTEMS**

Our femoral, tibial tray, tibial insert and patella button are absolutely unique to our systems. Under no conditions should these be matched with another manufacturer's component.

#### **EXTREMITY SYSTEMS**

All our lower and upper extremity systems are absolutely unique to our systems. Under no conditions should these be matched with another manufacturer's components.

#### STERILITY

**THESE DEVICES ARE SUPPLIED STERILE. DO NOT RE-STERILISE.  
IF DEVICE STERILITY IS COMPROMISED, CONTACT THE MANUFACTURER.**

#### CAUTION:

United States Federal Law restricts this device to sale by or on the order of a licensed physician